

Assembly Instructions For Wolf's Adjustable Mobile Shield

Stock Number 23111

Before starting assembly, check that parts 1 through 12 listed below have been included.

- 1) Snap the four casters (#7) into the sockets of the leg assembly (#6).
- 2) Assemble the leg assembly with the locking legs (#8) using four bolts (#5), leaving the bolts loose.
- 3) Put the unit on the floor, and check that the unit is square and the 4 casters are locked in place and the unit moves freely across the floor.
- 4) Insert two compression springs (#4) inside the leg assembly (#6), one compression spring on each side.
- 5) Tighten the 4 bolts (#5).
- 6) Insert the screen support (#1) with the pre-assembled screen (#2) into the leg assembly (#6). Adjust to the desired height and lock in place with the thumbscrews (#3).

Parts List

- 1 Screen support (1) #23111-02
- 2 Screen (1) # 23111-16 (Preassembled to 23111-02)
- 3 Thumbscrews (2) #23111-10
- 4 Compression springs (2) # 23111-11
- 5 Bolts (4) # 23000-07
- 6 Leg assembly (2) # 23111-08
- 7 Casters (4) # 23111-73
- 8 Locking legs (2) #23111-05
- 9 7/8" Hole plug (2) # 23111-03
- 10 Screen screws (3) # 23111-06
- 11 Plugs # 21178-36

